alma-c

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
After putting-on the ALMA-C underwear, first position the foam device on the perineum, then, while maintaining this device, pass a first strap.

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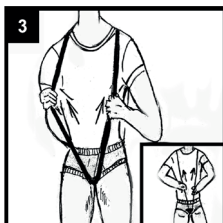
Then pass the second strap while holding the device on the perineum.

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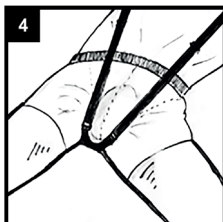Using the shoulder strap loops located on the front and the back, settle the adjustment in order to obtain the optimum pressure on the perineum.

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The foam device must be positioned and plated on the perineum from the beginning of the genital fellowships, the genital device of the user must be imperatively out of the system.

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ENGLISH

The adjustable solution for sphincter failure

1. What is sphincter failure ?
2. Solutions to date
3. You have just acquired ALMAC C
4. Description of ALMAC C and technical datas
5. Implementation procedure
6. Maintainance
7. ALMAC C benefits

What is sphincter failure ?

In the case of involuntary urine leakage as a result of efforts (sneeze, cough, lift a weight, ...), or without having to urinate, it is a passive losses of urine that will occur by walking or standing. It will appear more or less late in the day.

These continence issues will mainly occur to men, after surgery on prostate, and to women, after childbirth, sometimes when menopause, and more often will be linked to the age of the person.

After prostate surgery (especially in men after total prostatectomy), after childbirth in women or muscle wasting related to the aging of the person, there will be a relaxation of the tone of the perineal zone. This weakening of the pelvic floor, where muscles played a key role in controlling the bladder and sphincter, will cause a descent of the bladder and the urethra because they lost their support

There are three degrees of incontinence severity :

- 1st Degree : loss of urine when coughing, laughing, sneezing...
- 2nd Degree : loss of urine when lifting a weight, climbing stairs, walking...
- 3rd Degree : loss of urine when you are just standing.

Existing solutions to date

To date, the recommended solutions are :

- Perineal rehabilitation
- Surgery
- Urinary protections

And in a more invasive surgery: muscle graft or the placement of a urinary sphincter.

(For men, there is also the penis case (small reservoir collecting the urine) or the penis clip which is a small stop cock placed at the base of the penis).

Perineal rehabilitation

For some persons, it is often not really efficient despite its physiotherapy handling.

Surgery

Suburethral strips or ballons surgery, although they are in most cases good solutions, are invasive surgeries.

Urinary protections

Diapers, complete change, men's shells, etc..

They are unanimously recognized as traumatic by their users.

Regarding muscle graft and artificial sphincter, these costly and complex procedures are not always the appropriate solutions for mild or moderate incontinence including IUE.

IMPORTANT NOTICE

It is understood that we only speak of stress incontinences and urinary leaks due to age, prostatic problems and childbirth.

Must be excluded from here all severe incontinence due to diseases like Cancer, AVC, Parkinson, diabete, malformation, trauma, stone...

As well as functional or urge incontinence characterized by the inability to retain urine due to dysfunction of the lower urinary tract.

These bladder control problems need to be handled by the doctor.

Therefore, as soon as someone is suffering from incontinence, should first consult his urologist or gynecologist in order to rule out any pathology and conclude, after diagnosis, only to an incontinence linked to the effort.

(It should also be noted that UIE is not considered as a disease because there is no specific etiology. Also note that no drug is currently active for this problem)

You have just acquired ALMA C

This painful situation that you are experiencing either with an UIE or with passive urinary leaks has led us to seriously consider this problem and to develop a system providing an innovative solution.

After a development and many efficacy tests that our company launched the ALMA-C concept.

The product comes in a kind of adjustable, washable underwear, easy to wear and easy to remove, perfectly discreet, and putting on its own underwear.

ALMA-C : the adjustable solution

ALMA.C description - Implementation Procedure - Maintainance

ALMA-C is a type of single-sized underwear in polyester and Lycra material.

As an integrated device, it must be placed in contact with the perineal zone of the user (for men, from the beginning of the testicles).

This contact will be positioned in «clamped / tight» , ALMAC.C is equipped with a system of dual adjustment straps which will allow the device to play a role of «hammock» to the pelvic floor.

This «hammock» should then be placed in a «plated»

position on the perineum, and then, in order to obtain and balance the «tight», the shoulder straps will be adjusted for this purpose by means of sliding loops on the front and on the back.

ALMAC.C will therefore function as anatomical support, positioning the system, by raising the urethral bulb towards the bladder neck, will replace the role of the muscles of the pelvic floor, and will thereby remedy their insufficiency

WARNING

These shoulder straps are the most important in ALMA.C concept because in order to benefit from its maximum efficiency and to create an obstacle to the involuntary flow of urine, need to have the sensation of this permanent compression at the level of the perineum

Normally this strapless adjustment should be made only once, or upon the acquisition of the product.

Note for users: in case of long seated journeys by car or plane, the shoulder straps can be released during the trip (The sitting position can play the same role of compression on the pelvic area).

Maintainance : ALMA.C only requires hand washing with lukewarm water and mild soap.

ALMA.C benefits

- The system does not need any medical prescription and therefore does not require any prescription.
 - The system will mainly avoid using any protections(such as diapers, changes, plastic shells...) and will be an economical solution compared to the cost of these protections.
 - Easy maintainance
- ALMA.C is only an alternative on other solutions.
- The system will allow to avoid protections such as diapers, changes, shells
 - Replace the perineal rehabilitation exercises that have not yielded the expected results
 - - Finally help to find a temporary solution pending surgery, or for other reasons, avoid this surgery

Note : ALMA.C does not claim to be in any way a therapy for incontinence or sphincter insufficiency. Nevertheless, the results may be differents from one individual to another.

In the event of a defect in wearing the device or a problem of severe incontinence, our Company can not be responsible for a failure to achieve the expected results. ter insufficiency.